Potenza

ABRASIONE

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PHS DO BRASIL LTDA

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Potenza

ABRASIONE

INSTRUCTIONS FOR USE FOR PROFESSIONALS

PHS DO BRASIL LTDA

Guilherme Boldt Street, 210, District: Zona Industrial Norte, City: Joinville,

State: Santa Catarina, Country: Brazil - Zip Code: 89239-260

CNPJ: 17.910.389/0001-05 - BRAZILIAN INDUSTRY

E-MAIL: contato@phsdo brasil.com.br

Health Ministry's Operating Licence: 8.11992.7 (X401M5619X7L)

Notification at ANVISA number: 8.11992.79028

EU

REP

CINTERQUAL – Commerce Solutions

International Lda; Avenida Defensores de Chaves, Nº 4

Idea Spaces Office – 1000-117

Lisboa – Portugal +351 967 417 654

Product Commercial Name: POTENZA ABRASIONE.

Alternative Commercial Name: MICROABRASION PASTE

Technical Product Name: PASTES FOR DENTAL PROPHYLAXIS AND POLISHING.

READ ALL THE INFORMATION IN THIS INSTRUCTION FOR USE CAREFULLY BEFORE APPLYING THE PRODUCT.

KEEP THIS INSTRUCTION UNTIL THERE IS NO FURTHER INTERACTION OF THE PRODUCT WITH YOUR PATIENT.

Technical Support

0800-000-3780

www.phsgroup.com.br

BASIC COMPOSITION:

Hydrochloric acid, silicon carbide microparticles, thickener, and ultrapure water.

PRODUCT DESCRIPTION:

POTENZA ABRASIONE is a dental microabrasion paste composed of acid and abrasive microparticles, intended for the controlled chemical and mechanical abrasion of dental enamel. The product presents a viscous consistency, thixotropic characteristic, sandy appearance, and lead-gray color, allowing better control during clinical application. It is supplied in a ready-to-use syringe.

PRODUCT INDICATION:

POTENZA ABRASIONE is indicated for the controlled removal of superficial dental enamel stains resulting from fluorosis, enamel hypoplasia, and other superficial alterations associated with pigment accumulation. The product acts through chemical and mechanical microabrasion, promoting the removal of the stained superficial enamel layer.

PRECAUTIONS:

- Avoid direct contact of POTENZA ABRASIONE with skin, mucosa, and soft tissues. In case of accidental contact, immediately rinse the affected area thoroughly with water;
- During product use, adequate isolation of the operative field is recommended due to the presence of acid in the formulation;
- The professional, patient, and assistant must use appropriate personal protective equipment, including gloves, mask, and protective eyewear;
- Protect the patient with a waterproof drape and protective eyewear throughout the procedure;
- After use, remove and properly discard the applicator tip used during the session. Do not reuse disposable components;
- After use, release the internal syringe pressure by slightly pulling the plunger backward and immediately close it with the original cap;
- In case of eye contact, rinse immediately with plenty of water and seek medical assistance, taking these instructions for use with you;
- Do not use the product without supervision and application control by a qualified professional.

CONTRAINDICATIONS:

- POTENZA ABRASIONE is contraindicated for patients with a history of hypersensitivity or allergy to any components of the formulation;
- It is contraindicated for the removal of deep stains or discolorations requiring excessive enamel wear;
- Do not use on teeth with severely compromised enamel, pronounced hypoplasia, or insufficient enamel thickness;

- It is contraindicated for patients with active periodontal diseases or oral conditions that contraindicate abrasive procedures;
- Do not use in patients with incomplete root formation.

POSSIBLE SIDE EFFECTS:

- Temporary dental hypersensitivity may occur after treatment. In such cases, the use of desensitizing agents based on sodium fluoride and potassium nitrate is recommended. In persistent or severe cases, clinical evaluation by a dentist is recommended;
- Accidental contact of the product with skin, mucosa, or soft tissues may cause temporary local irritation;
- In case of allergic reaction or hypersensitivity to any formulation component, discontinue use immediately and advise the patient to seek medical assistance, taking these instructions for use with them.

WARNING:

- Observe the product expiration date;
- Do not use the product after the expiration date;
- Keep out of reach of children;
- Product intended exclusively for professional dental use;
- Before use, verify package integrity and product appearance. In case of abnormalities, do not use and contact the manufacturer;
- This product must be handled exclusively according to these instructions for use;
- The manufacturer is not responsible for damages caused by improper, incorrect, or non-compliant use with these instructions;
- It is the professional's responsibility to previously evaluate the clinical indication and suitability of the product for each patient;
- In case of a serious incident related to product use, immediately notify the manufacturer and the competent health authority;
- Do not reuse the empty packaging for other purposes;
- Reuse of the microabrasion paste may result in microbiological contamination and risk of cross-contamination between patients;
- Reuse of the product may alter its physicochemical characteristics, including active component concentration, viscosity, abrasiveness, and clinical performance;
- Do not reuse applicator tips or disposable components;
- For disposal of the product and its components after use, follow applicable local regulations regarding dental and healthcare waste.

INSTRUCTIONS FOR USE:

Step 1: Before indicating the microabrasion procedure, ensure that the discoloration or stain is restricted superficially to the dental enamel. Transillumination examination may assist in diagnosis and clinical planning;

Step 2: Perform adequate rubber dam isolation to avoid product contact with soft tissues and minimize the risk of accidental ingestion during the procedure;

Step 3: Slightly moisten the dental surface to be treated and apply a small amount of POTENZA ABRASIONE over the stained area. The application may be performed according to one of the following techniques:

Mechanical application – rubber cup at low speed:

Apply light to moderate pressure using intermittent movements for approximately 10 seconds.

Manual application – plastic spatula:

Apply moderate pressure using intermittent movements for approximately 10 seconds.

Step 4: After each application, rinse the treated area thoroughly for at least 10 seconds and completely aspirate the contents. Clinically evaluate the obtained result and, if necessary, repeat the procedure. The application may be repeated up to 15 times in the same session or in different sessions, according to professional evaluation;

Step 5: After completing the procedure, remove the rubber dam isolation and polish the treated surface using a felt disc and polishing paste (e.g., Potenza Specchi PRO). Then rinse and dry the area;

Step 6: To conclude the session, apply a sodium fluoride-based desensitizing agent (e.g., Potenza Esente 2%) for 5 minutes.

CONSERVATION AND STORAGE:

- Keep the product in its original packaging;
- Protect from the incidence of direct sunlight;
- Store the product at temperatures from 0 °C to 30 °C (32 °F and 86 °F).

TRANSPORT CONDITIONS:

- The product can be transported at temperatures between -5 °C to 50 °C (23 °F and 122 °F).
- Protect from the incidence of direct sunlight.

MANUFACTURING DATE: SEE PACKAGING.

EXPIRATION DATE: 2 YEARS AFTER THE MANUFACTURE DATE.

EXCLUSIVELY FOR USE IN DENTAL OFFICES. FOR USE ONLY BY DENTAL PROFESSIONALS.

MAKE SURE THE INSTRUCTION FOR USE MATCHES THE PRODUCT TO BE USED.

TO ACCESS A PRINTED VERSION OF THE INSTRUCTION FOR USE, CONTACT THE COMPANY'S TECHNICAL SUPPORT.

KEEP THE PACKAGE CLOSED WHEN NOT IN USE.

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