Potenza

CLOUD CLEAN

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PHS DO BRASIL LTDA

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Potenza

CLOUD CLEAN

INSTRUCTIONS FOR USE FOR PROFESSIONALS

PHS DO BRASIL LTDA

Guilherme Boldt Street, 210, District: Zona Industrial Norte, City: Joinville,

State: Santa Catarina, Country: Brazil - Zip Code: 89239-260

CNPJ: 17.910.389/0001-05 - BRAZILIAN INDUSTRY

E-MAIL: contato@phsdo brasil.com.br

Health Ministry's Operating Licence: 8.11992.7 (X401M5619X7L)

Notification at ANVISA number: 8.11992.79035

CINTERQUAL – Commerce Solutions

Internacional Lda; Avenida Defensores de Chaves, Nº 4

Idea Spaces Office – 1000-117

Lisboa – Portugal +351 967 417 654

Product Commercial Name: POTENZA CLOUD CLEAN.

Alternative Commercial Name: CLEANING FOAM FOR TRAYS AND REMOVABLE ORTHODONTICS.

Technical Product Name: MATERIAL FOR CLEANING REMOVABLE ORTHODONTIC APPLIANCES.

READ ALL THE INFORMATION IN THIS INSTRUCTION FOR USE CAREFULLY BEFORE APPLYING THE PRODUCT.

KEEP THIS INSTRUCTION UNTIL THERE IS NO FURTHER INTERACTION OF THE PRODUCT WITH YOUR PATIENT.

Technical Support

0800-000-3780

www.phsgroup.com.br

BASIC COMPOSITION:

Water, hydrogen peroxide, thickener, foaming agent, flavoring.

PRODUCT DESCRIPTION:

POTENZA CLOUD CLEAN is a cleaning foam for removable oral devices (aligners, orthodontic retainers and mouthguards). Cleaning of mobile intraoral devices is done through the action of surfactant (sodium lauryl sulfate) and oxidative (hydrogen peroxide) agents.

PRODUCT INDICATION:

POTENZA CLOUD CLEAN is intended for cleaning removable intraoral devices, such as: invisible aligners, orthodontic retainers and mouthguards.

PRECAUTIONS:

- Follow the instructions for use;
- Do not use if the packaging is damaged;
- Do not use after the expiration date;
- Adult supervision is required for children under 12 years of age;
- Avoid contact with eyes, skin or hair. If it occurs, wash abundantly and monitor until remission of any signs or symptoms;
- Do not leave the product within the reach of children or people not qualified for its use;
- Right after using the product, keep its packaging properly closed;
- Before reusing the removable intraoral device, make sure to remove the entire content of the product by rinsing it under running water.

CONTRAINDICATIONS:

- The product must not be used in the mouth;
- Do not use the product on patients who are allergic to any of the components of the formula.

POSSIBLE SIDE EFFECTS:

- If the product is used as directed, no side effects are expected.

WARNING:

- Avoid contact of the product with mucosa, skin, eyes, and hair;
- Store the product away from sunlight and heat sources;
- Keep out of reach of children or persons not qualified to use the product;
- Do not reuse the empty packaging for other purposes. Dispose of the product according to local regulations.

INSTRUCTIONS FOR USE:

To obtain the desired effects, the intraoral device must be placed in an appropriate container (e.g., case or container), and the foam should be applied so as to cover its entire surface. Allow the product to act for 15 minutes. After this period, thoroughly rinse the intraoral device under running water before reuse.

CONSERVATION AND STORAGE:

- Store the product at room temperature 0 °C to 30 °C (32 °F to 86 °F);
- Protect it from moisture and direct sunlight.

TRANSPORT CONDITIONS:

- The product can be transported at temperatures between -5 °C and 50 °C (23 °F and 122 °F).

MANUFACTURING DATE: SEE PACKAGING.

EXPIRATION DATE: 2 YEARS AFTER THE MANUFACTURE DATE.

MAKE SURE THE INSTRUCTION FOR USE MATCHES THE PRODUCT TO BE USED.

TO ACCESS A PRINTED VERSION OF THE INSTRUCTION FOR USE, CONTACT THE COMPANY'S TECHNICAL SUPPORT.

KEEP THE PACKAGE CLOSED WHEN NOT IN USE.

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