Potenza

BIANCO PRO SS 38%

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H2O2 ON REFIL

PHS DO BRASIL LTDA

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H2O2 ON REFIL

PHS DO BRASIL LTDA

Guilherme Boldt Street, 210, District: Zona Industrial Norte, City: Joinville,

State: Santa Catarina, Country: Brazil - Zip Code: 89239-260

CNPJ: 17.910.389/0001-05 - BRAZILIAN INDUSTRY

E-MAIL: contato@phsdobrasil.com.br

Health Ministry's Operating Licence: X401M5619X7L (8.11992.7)

Notification at ANVISA number: 81199270031

CINTERQUAL – Commerce Solutions International Lda

Avenida Defensores de Chaves, Nº 4 – Idea Spaces Office – 1000-117

Lisboa – Portugal +351 967 417 654

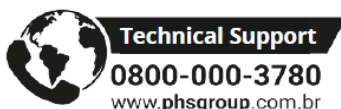
Product Commercial Name: POTENZA BIANCO PRO SS 38% H2O2 ON REFIL.

Alternative Commercial Name: POTENZA BIANCO PRO SS DENTAL WHITENING FOR USE IN OFFICES.

Technical Product Name: DENTAL WHITENING AGENTS

READ ALL THE INFORMATION IN THIS INSTRUCTION FOR USE CAREFULLY BEFORE APPLYING THE PRODUCT.

KEEP THIS INSTRUCTION UNTIL THERE IS NO FURTHER INTERACTION OF THE PRODUCT WITH YOUR PATIENT.

BASIC COMPOSITION:

POTENZA BIANCO PRO SS 38% H₂O₂ ON Refil: Hydrogen peroxide, humectant, thickeners, neutralizing agent, coloring agent and ultrapure water.

PRODUCT DESCRIPTION:

POTENZA BIANCO PRO SS 38% H₂O₂ ON Refil is a professional dental whitening agent intended for in-office application under the supervision of a dental surgeon.

The system is supplied in two syringes: one containing hydrogen peroxide and the other containing a thickening agent associated with a humectant agent. The phases are mixed immediately before use, providing adequate viscosity, stability and dispersion of the whitening agent.

Homogenization occurs through the connection of the syringes using a Luer Lock threaded system, resulting in an intense blue thixotropic gel developed to facilitate visualization, clinical application control and product removal at the end of the procedure.

The gel presents suitable consistency for application onto the dental surface, minimizing flow into undesired areas. Application is performed using applicator tips directly attached to the syringe.

PRODUCT' INDICATION:

POTENZA BIANCO PRO SS 38% H₂O₂ ON Refil is indicated for in-office dental whitening procedures under the supervision of a dental surgeon.

The product may be used on vital and non-vital teeth exclusively through the external whitening technique in cases presenting color alteration resulting from intrinsic or extrinsic factors.

The whitening agent may also be used as an adjunctive treatment in mild to moderate cases of dental staining caused by fluorosis or tetracycline and its derivatives.

In severe discoloration cases, treatment may be complemented with additional sessions or associated with supervised at-home whitening techniques, according to the clinical evaluation of the responsible professional.

PRECAUTIONS:

- Before starting treatment, perform a detailed anamnesis and clinical evaluation of the patient, verifying the presence of factors that may contraindicate or limit dental whitening;
- If the patient reports previous dental hypersensitivity during routine activities (e.g., ingestion of cold liquids, exposure to cold air or chewing pressure) without apparent clinical cause, it is recommended to previously perform a desensitizing protocol with Potenza Esente 2% and Potenza Esente GLU-HEMA;
- In cases of significant gingival recession, exposed dentin, cracks, infiltrated restorations or other conditions that may favor sensitivity or contact of the whitening agent with sensitive structures, prior treatment or proper protection of the region is recommended before whitening;

- Dental surfaces must be clean and free from calculus, bacterial plaque or residues. If necessary, perform prophylaxis and/or scaling before whitening, preferably with a minimum interval of 5 days prior to the session;
- It is essential to avoid contact of the whitening gel with soft tissues, mucosa, skin or eyes, since high concentration hydrogen peroxide may cause irritation, chemical burns, transient discomfort or more severe tissue injuries. Ensure correct application of the gingival barrier and use an appropriate lip retractor throughout the procedure;
- In case of accidental contact of the whitening gel with soft tissues, immediately remove excess product, apply the neutralizing solution and rinse thoroughly with water. If irritation, pain or significant tissue alteration persists, interrupt the procedure and adopt appropriate clinical conduct;
- If the patient presents intense hypersensitivity during the session, immediately interrupt treatment and apply Potenza Esente 2% desensitizer for approximately 10 minutes;
- The product must not be used under local anesthesia, since monitoring the sensitivity reported by the patient is important for early identification of possible complications;
- Prior acid etching of enamel is neither necessary nor recommended before use of the product;
- Some patients may not be eligible for in-office whitening or may present better clinical response with supervised at-home whitening protocols using lower concentration whitening agents;

IMPORTANT:

- Whitening treatment must be performed exclusively by a dental surgeon or under direct professional supervision;
- Treatment safety and effectiveness depend on correct clinical indication, proper product application and full compliance with the instructions for use;
- Whitening response may vary significantly among patients according to individual factors such as type and intensity of pigmentation, dental permeability, age, structural characteristics of enamel and dentin, eating habits and clinical history;
- In some cases, additional in-office whitening sessions or association with supervised at-home whitening techniques may be necessary to achieve the desired result;
- Restorations, crowns, veneers and other restorative materials do not undergo whitening and may require aesthetic replacement after treatment completion.

CONTRAINDICATIONS:

Before starting whitening treatment, a detailed anamnesis and complete clinical evaluation of the patient must be performed, considering habits, medical and dental history, systemic conditions, chronic or acute diseases and local conditions that may interfere with the safety or effectiveness of the procedure.

Transient clinical alterations must be previously treated before performing whitening procedures. Some conditions absolutely contraindicate the procedure.

A) TEMPORARY CONTRAINDICATIONS – treat before starting whitening:

- Active dental caries;
- Infiltrated, defective or poorly adapted restorations;
- Dental cracks, fractures or structural defects of enamel and dentin;
- Presence of bacterial plaque, dental calculus or inadequate oral hygiene;
- Gingivitis, periodontitis or other uncontrolled periodontal alterations;
- Teeth with questionable vitality;
- Untreated endodontic problems;
- Extensive gingival recession with significant root exposure;
- Severe dental hypersensitivity without clinical control;
- Oral lesions, ulcerations or soft tissue irritation;
- Recent periodontal surgeries or gingival tissues undergoing healing process.

B) ABSOLUTE CONTRAINDICATIONS – do not perform treatment:

- Patients with history of hypersensitivity or allergy to any component of the formulation;
- Pregnant or breastfeeding women;
- Individuals under 15 years of age;
- Teeth with incomplete root formation;
- Teeth with severe mobility or severe bone loss;
- Teeth affected by amelogenesis imperfecta or dentinogenesis imperfecta;
- Severe fluorosis or intense staining caused by tetracycline and its derivatives;
- Patients with severe decompensated systemic diseases or undergoing medical treatment contraindicating the procedure, according to professional evaluation;
- Clinical situations preventing proper field isolation or safe control of contact between the whitening agent and soft tissues.

The product must not be used under local anesthesia, since monitoring the sensitivity reported by the patient is important for early identification of possible complications during the procedure.

POSSIBLE SIDE EFFECTS:

- It is common for the patient to present some degree of dental hypersensitivity during the whitening session or within the first 48 hours after the procedure. This effect is related to the mechanism of action of hydrogen peroxide itself and may vary significantly among patients regarding intensity and duration. The use of desensitizing agents such as Potenza Esente 2% and

Potenza Esente GLU-HEMA may help reduce discomfort. In cases of intense or persistent sensitivity, treatment interruption and reassessment of treatment continuity are recommended;

- Accidental contact of the whitening gel with soft tissues may cause irritation, burning sensation, transient discomfort or temporary whitish areas resulting from the chemical action of hydrogen peroxide. In such cases, immediately remove the product, apply the neutralizing solution and rinse thoroughly with water;
- Dental whitening may highlight pre-existing areas of hypocalcification, enamel hypoplasia or other structural alterations due to temporary dehydration of teeth or differential whitening of such regions. In most cases, these alterations become less noticeable after natural tooth rehydration or surface remineralization, which usually occurs within a few hours or days after the session;
- If whitish areas or aesthetic alterations persist after the rehydration period, it is recommended to temporarily suspend new sessions and clinically reassess the case, as remineralizing treatment or restorative intervention may be necessary;
- Some patients may present uneven whitening among different thirds of the teeth. This condition is related to physiological and structural characteristics of the dental element, especially the greater dentin thickness in the cervical region, naturally more pigmented than incisal regions;
- In some cases, transient pulpal irritation, pressure sensation or mild discomfort may occur after the procedure, usually reversible within a short period;
- Restorations, crowns, veneers and other restorative materials do not undergo whitening and may present color differences in relation to whitened teeth after treatment.

WARNING:

- Observe the expiration date indicated on the packaging. Do not use the product after expiration;
- Keep out of reach of children;
- Product for professional dental use only;
- Before use, verify package integrity and product appearance. In case of color alterations, consistency changes, leakage, excessive syringe pressure or any other abnormality, discontinue use and contact the manufacturer;
- High concentration hydrogen peroxide may cause irritation, chemical burns and severe ocular injuries in case of improper contact;
- The product must not be ingested, injected or used for purposes other than those described in these instructions for use;
- The product must not be used under local anesthesia, since monitoring the sensitivity reported by the patient is important for early identification of possible complications;
- After phase mixing, the product must be used immediately during the same clinical session. Do not reuse leftover whitening gel;
- Do not reuse packaging, syringes, applicator tips or empty containers for any purpose;

- The manufacturer is not responsible for damages resulting from improper use, incorrect handling, inadequate storage or use contrary to these instructions for use;
- It is the professional's responsibility to previously evaluate the patient's clinical conditions and verify whether the product is suitable for the intended purpose;
- The product must be stored and transported according to the conditions specified in these instructions for use;
- For disposal of residues, leftover product and packaging, follow applicable local legislation.

INSTRUCTIONS FOR USE:

Before using the product, read these instructions for use in full. Perform a detailed anamnesis and complete clinical examination of the patient, previously resolving any conditions that may contraindicate or interfere with the safety and effectiveness of the whitening treatment (see "Precautions" and "Contraindications").

Throughout the procedure, continuously monitor the patient for the occurrence of hypersensitivity, tissue irritation or accidental contact of the whitening gel with soft tissues. If any adverse event occurs, immediately interrupt the procedure and adopt the measures recommended in these instructions for use.

Step 1: Select and record the initial tooth shade using an appropriate shade guide. Photographic documentation prior to treatment is also recommended;

Step 2: Apply Potenza Idrata moisturizing base to the patient's lips, especially at the lip commissures, in order to hydrate, protect tissues and provide greater comfort during the procedure. Then position an appropriate lip retractor to facilitate access to the teeth and minimize accidental contact of the gel with soft tissues;

Step 3: Perform the pre-whitening desensitizing protocol:

3.1. Apply Potenza Esente 2% desensitizer to the buccal surface of the teeth to be whitened, maintaining the product in contact for approximately 10 minutes. During this period, perform gentle friction movements using a disposable microapplicator. At the end, rinse thoroughly with water/air spray while simultaneously using suction to avoid product ingestion, and dry the teeth;

3.2. If the patient presents exposed dentin or gingival recession with root exposure, apply Potenza Esente GLU-HEMA using a disposable brush directly onto the exposed region, avoiding contact with the gingival margin. Allow it to act for approximately 45 seconds, rinse thoroughly and dry the surfaces;

Step 4: Carefully dry the gingival tissues and apply the Potenza Blocco light-cured gingival barrier over the marginal gingiva of the teeth to be whitened. The barrier should present approximately 1 mm thickness and approximately 5 mm width, completely covering the gingival papillae and approximately 0.5 mm of the cervical enamel region.

Light-cure the gingival barrier for 20 to 30 seconds for each group of 3 teeth. After polymerization, confirm that the barrier is rigid, intact and properly adhered to the gingiva.

IMPORTANT: Using a dental mirror, carefully observe the region from incisal to cervical, verifying whether there is any exposed gingival area. If necessary, perform corrections before applying the whitening gel;

Step 5: Remove the protective caps from the syringe containing hydrogen peroxide and from the syringe containing the thickener. Connect one syringe to the other by screwing them together until complete locking is achieved. Alternately push the syringe plungers (always from the peroxide side toward the thickener side) 14 to 20 times (7 to 10 movements in each direction), slowly and continuously.

After mixing, return the mixed content to the syringe that originally contained the hydrogen peroxide (internal threaded end), carefully disconnect the syringes and attach the applicator tip to the syringe now containing the mixed gel.

IMPORTANT: After mixing, the whitening gel must be used immediately during the same clinical session. Do not reuse leftover product;

Step 6: Apply the whitening gel onto the buccal surface of the teeth using sweeping motions, completely covering the areas to be whitened, including interproximal regions and slightly extending onto the incisal/occlusal surfaces. The applied layer should present approximately 1 mm thickness, avoiding excess material and flow into adjacent tissues;

Step 7: Allow the product to act for 25 to 30 consecutive minutes. During the action period, continuously monitor the patient and eliminate possible oxygen bubbles using an explorer probe or microapplicator.

POTENZA BIANCO PRO SS 38% H₂O₂ ON Refil does not require gel replacement during the clinical session.

The use of auxiliary light sources is not necessary for the effectiveness of the product and may increase dental sensitivity and/or pulpal heating.

Do not exceed 30 consecutive minutes of gel contact with the dental surface per clinical session;

Step 8: After the recommended time, carefully remove the whitening gel using an aspirating cannula (e.g., endodontic or surgical cannula) and gauze. Then thoroughly rinse the dental surfaces with water while simultaneously using suction to avoid ingestion of product residues;

Step 9: Carefully remove the gingival barrier using an explorer probe, gently pulling it in the anterior direction. Rinse the dental surface again with water/air spray while always using simultaneous suction;

Step 10: Perform the final polishing of the teeth using felt discs associated with Potenza Specchi PRO diamond polishing paste.

The final whitening result may be better visualized after approximately 7 days, which is the period necessary for dental rehydration.

Normally, 2 to 3 clinical sessions are necessary, with a minimum interval of 7 days between them, although the protocol may be adjusted according to the patient's individual clinical evaluation.

It is recommended to avoid strongly pigmented

GENERAL SAFETY INSTRUCTIONS:

- Product handling must be performed exclusively in a dental office by a dental surgeon or under direct professional supervision;
- The professional and assistant must use all personal protective equipment (PPE) recommended for dental procedures, including gloves, mask, protective eyewear and appropriate protective clothing;
- The patient must wear protective eyewear, disposable cap and protective covering over clothing throughout the procedure;
- The syringe containing hydrogen peroxide may be under pressure or present leakage due to improper storage or transportation conditions. Before opening the package, protect hands and eyes and carefully verify the presence of leakage or package alterations;
- Avoid contact of the product with skin, mucosa, eyes, clothing or surfaces not intended for the clinical procedure;
- Instruct the patient not to ingest the product or residues resulting from the whitening procedure;
- In case of accidental contact of the whitening gel with skin or intraoral mucosa, immediately remove excess product, apply a few drops of the neutralizing solution onto the affected region and rinse thoroughly with water;
- The neutralizing solution acts only on peroxide residues still present on the surface and does not reverse damage already caused by the chemical action of the whitening agent;
- In case of accidental contact with the eyes, rinse immediately with plenty of water for several minutes and refer the patient for medical evaluation, taking these instructions for use;
- If the patient presents allergic reaction, severe irritation, chemical burn, persistent pain or any significant adverse reaction, immediately interrupt treatment and adopt appropriate clinical conduct;
- The product must not be ingested, injected or used for purposes other than those described in these instructions for use;
- After phase mixing, the whitening gel must be used immediately during the same clinical session. Leftover product or residues must not be reused;
- Do not reuse applicator tips, empty syringes or packaging for any other purpose;
- For disposal of residues, leftover product and packaging, follow applicable local legislation.

CONSERVATION AND STORAGE:

- Maintain the product in its original package;
- Protect it from direct sunlight;
- Keep it in temperatures of 0 °C to 30 °C (32 °F to 86 °F).

TRANSPORT CONDITIONS:

- Product must be transported in temperatures between 0 °C and 30 °C (32 °F to 86 °F);
- Protect it from direct sunlight.

MANUFACTURING DATE: SEE PACKAGING.

EXPIRATION DATE: 2 YEARS AFTER THE MANUFACTURE DATE.

SALES UNDER PRESCRIPTION.

EXCLUSIVELY FOR USE IN DENTAL OFFICES. FOR USE ONLY BY DENTAL PROFESSIONALS.

MAKE SURE THE INSTRUCTION FOR USE MATCHES THE PRODUCT TO BE USED.

TO ACCESS A PRINTED VERSION OF THE INSTRUCTION FOR USE, CONTACT THE COMPANY'S TECHNICAL SUPPORT.

KEEP THE PACKAGE CLOSED WHEN NOT IN USE.

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